

A REVOLUTION IN HbA1c MEASUREMENT



Affordable A1c
testing

Semi-automated
analyser

Easy to use

Results in just
4 minutes

EASY TO USE

- User-friendly features minimise training time
- Step by step instructions on display
- User selectable language
- Just 4 µl blood required from finger prick or venous sample
- Innovative blood collector allows easy and consistent sampling

FAST AND ACCURATE

- Results within 4 minutes
- Uses boronate affinity methodology which is widely recognised as interference-free
- Measuring range: 4-15% A1c DCCT
- Imprecision: CV < 3% at 7% A1c DCCT
- Unaffected by Hb variants, which do not result in reduced erythrocyte life span
- Traceable to the IFCC reference method

EFFICIENT DATA HANDLING

- Barcode reader to scan calibration data, patient and operator ID
- Stores up to 7,000 results
- User selectable dual reporting. User can select % DCCT, IFCC mmol/mol, eAG mg/dl or eAG mmol/l
- USB port
- Optional label printer available

ERGONOMIC DESIGN

- Compact: 95 mm(h) x 205 mm(w) x 135 mm(d)
- Lightweight: 0.7 kg



1 Scan lot calibration barcode. Remove foil from cartridge and insert into analyser.



2 When prompted press reagent bead into cartridge using the reverse end of the sampling tool.



3 Insert sample tool into cartridge and snap off handle.



4 Close analyser lid. Result displayed within 4 minutes.

Affordable, accurate and reliable **HbA1c measurement**



Distributed by

Manufacturer

Quotient Diagnostics Ltd
 Russell House, Molesey Road,
 Walton-on-Thames, Surrey, UK
 KT12 3PE
Tel. +44 (0)1932 220 124

Sales

EKF Diagnostics Holdings plc
 Avon House, 19 Stanwell Road,
 Penarth, Cardiff, UK CF64 2EZ
Tel. +44 (0)29 20 710 570
www.ekfdiagnostics.com



ISO13485:
 2003 Certified.
 Registration ref:
 MD 505902



SCIENCE MADE SIMPLE