


1 Unit

GAMSCO

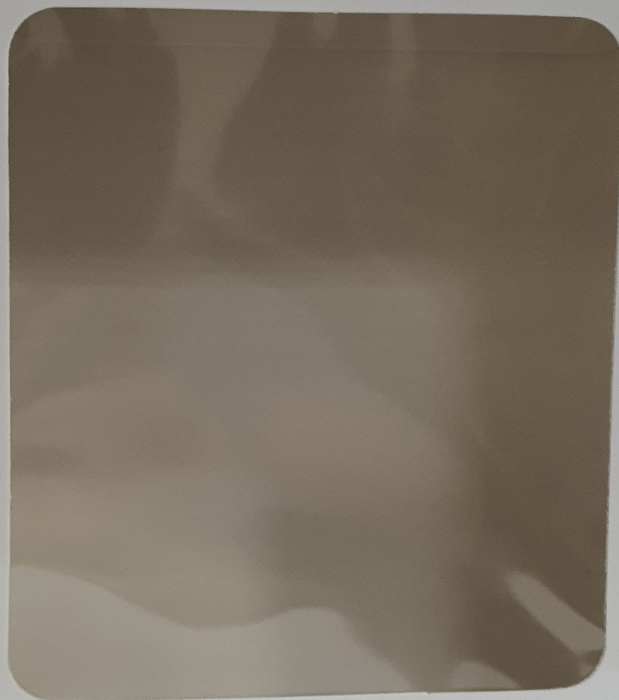

GAMSCO

Plasma Container Carton

Instructions

This carton should be used for freezing, storing and thawing plasma in Transfer Pack Container (300-400 mL capacity).

- Loading:** Form box and tuck in one end. Place plastic plasma container in box with label facing towards window. Fold over top of container and tuck in top flap of carton.
- Freezing:** Place this carton with its plasma container in freezer. The carton may be frozen or stored in any position. Faster freezing is obtained by laying flat on freezing shelf.
- Storage:** Store frozen plasma in this carton in the freezer compartment. Once contents are frozen the carton helps reduce the possibility of cracking the frozen plasma container.
- Thawing:** Remove carton from freezer and place in plastic overwrap. Seal overwrap tightly to prevent any moisture from entering product, and place in constant temperature water bath at 37°C until thawed.

GAMSCO

General Analysis for Medical Services Company

Tel : + 966 - 3 - 8394085 Fax: + 966 - 3 - 8394084

Dammam - Saudi Arabia

www.gamscomed.com

Serial No. _____

Expiration Date _____

Blood Group _____ Rh Type _____

Volume _____