



H. pylori Antibody Rapid Test

For laboratory professional use

As an aid in the diagnosis of H. pylori infection in symptomatic or asymptomatic individuals

H. pylori Antibody Rapid Test is intended for the qualitative presumptive detection of IgG antibodies to *Helicobacter pylori* in human whole blood, serum, or plasma specimens.



Product Information

Information Type	Specimen type	Detection	Format	Package	Time to results	Shelf life & Storage Conditions
Product Detail	Whole blood, serum or plasma	<i>Helicobacter pylori</i> antibodies	Device	25/40 pcs/box	10 minutes	24 months at 2-30 C


Test Procedure

1 For Serum or Plasma specimens



Transfer 3 drops of serum or plasma (approximately 75 μ L) to the specimen well (S).

For Whole Blood specimens

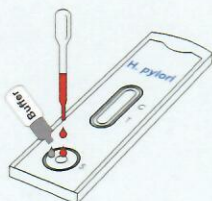

Allow 2 drops of whole blood (approximately 50 μ L) to the specimen well (S), then add 1 drop of buffer.

Notable Features

- Easy to Use
- Fast Detection: Result in 10 minutes.
- Rapid & accurate test results
- High Accuracy.

2



Read result at 10 minutes.

3



Positive



Negative



Invalid

Assure Tech. (Hangzhou) Co., Ltd.
Building 4, No. 1418-50, Moganshan Road,
Gongshu District, Hangzhou, 310011 Zhejiang, China

contact@diareagent.com
www.diareagent.com
400-853-5577

